







*New Options Sports*  
 "The New Standard in Sports Medicine"  
 "Made Well in the USA"

# Knee Mate™

## Wrap Around Knee Orthosis

### Fitting Instructions

1. Center hole on back of the knee.
2. Wrap top portion around thigh and close Velcro then close lower portion around calf
3. Wrap strap around back of leg all the way around to the front. Thread strap through buckle, adjust to comfortable compression and close Velcro
4. Support is now properly fit

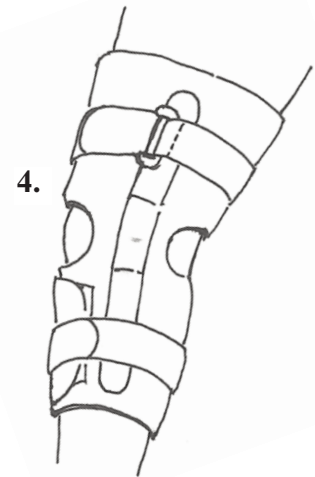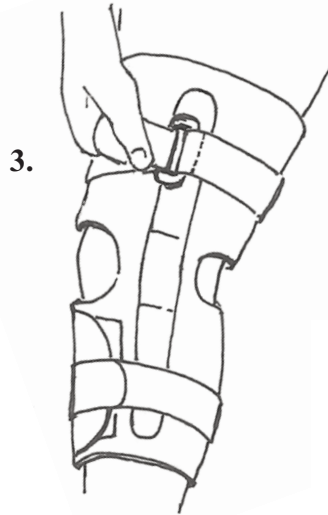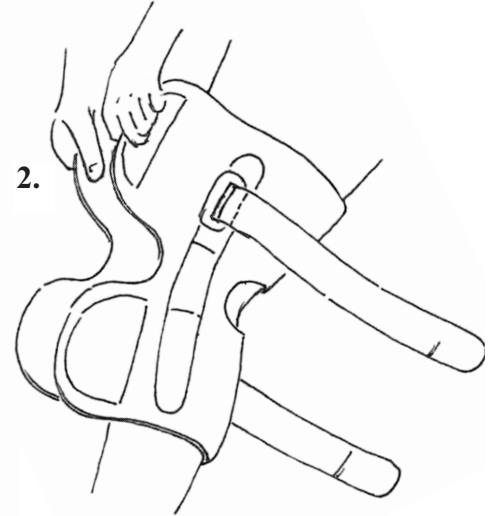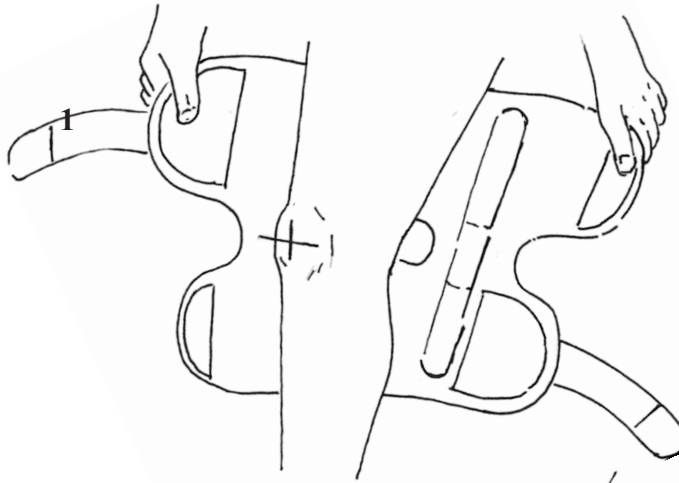

### Washing Instructions:

Remove hinges from pockets. Hand wash in cold water with mild detergent. NO chlorine bleach. Air dry. Do NOT put in dryer.

SIZE	KNEE CIRCUMFERENCE	6" ABOVE KNEE
XSMALL	12" - 13"	16" - 19"
SMALL	13" - 14"	17" - 20"
MEDIUM	14" - 15"	18" - 21"
LARGE	15" - 16"	19" - 22"
XLARGE	16" - 18"	20" - 24"
2XLARGE	18" - 20"	22" - 26"
3XLARGE	20" - 22"	24" - 28"
4XLARGE	22" - 24"	26" - 30"
CUSTOM	24" & UP - PLEASE PROVIDE MEASUREMENTS	

**1850 DIPLOMAT DRIVE, #100, FARMERS BRANCH, TEXAS 75229**  
**OFFICE (800) 872-5488 FAX (800) 455-5488**

