




# Knee Mate KC65-HT

## 17" Wrap Around Knee Orthosis

### Fitting Instructions

1. Center hole on back of the knee.
2. Wrap top portion around thigh and close Velcro then close lower portion around calf.
3. Wrap the bottom strap around back of leg all the way around to the front.  
Thread strap through D-ring buckle. Adjust to comfortable compression and close Velcro. Repeat for the strap above until all four straps are closed, finishing with the top strap.
4. Support is now properly fit

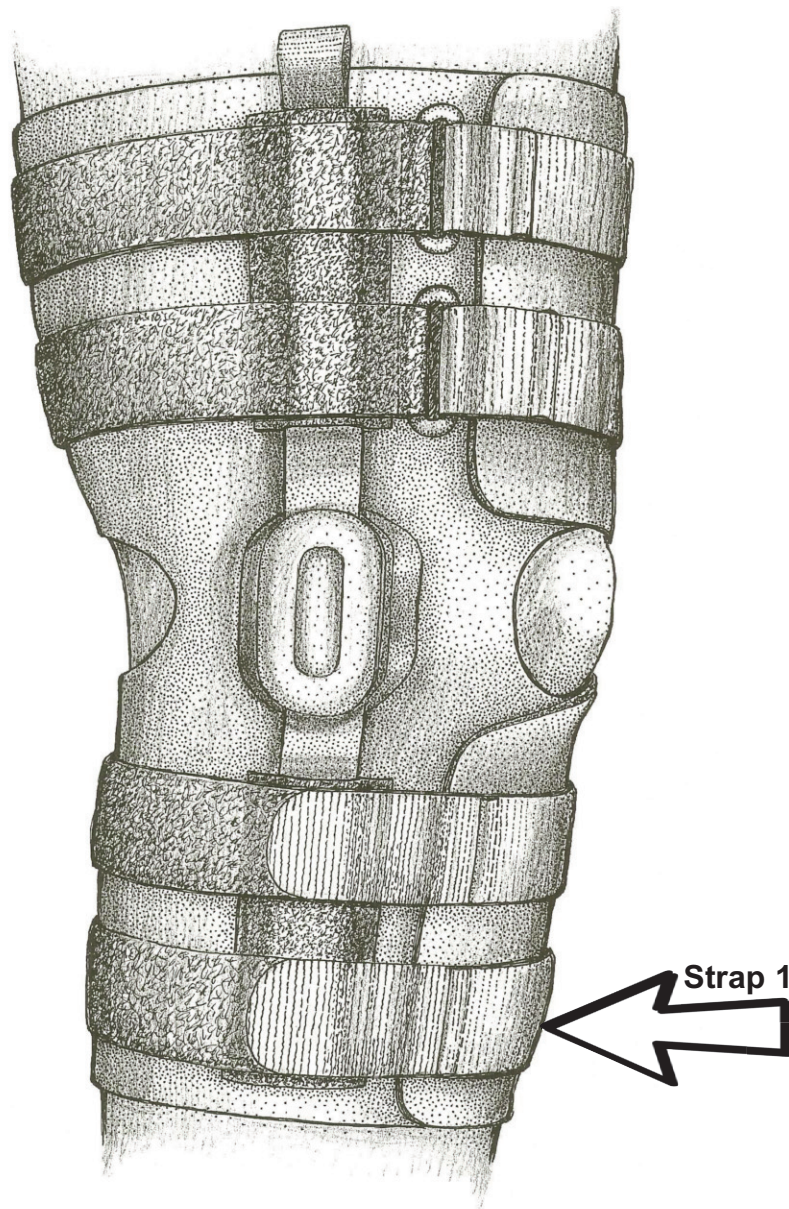

### Washing Instructions:

Remove hinges from pockets. Hand wash in cold water with mild detergent. NO chlorine bleach. Air dry. Do NOT put in dryer.

1850 DIPLOMAT DRIVE, #100, FARMERS BRANCH, TEXAS 75229  
OFFICE (800) 872-5488 FAX (800) 455-5488

